




It's simply breathtaking

## INSTRUCTIONS


1

Remove the plastic protective cover from the sampling device.

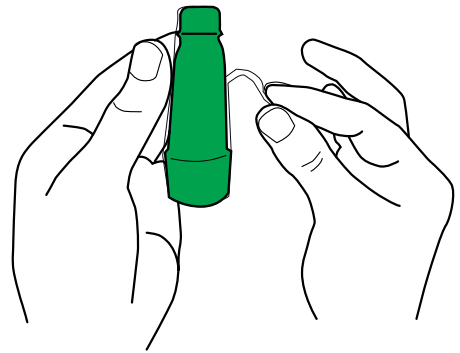


2

Remove the caps from the sampling device.

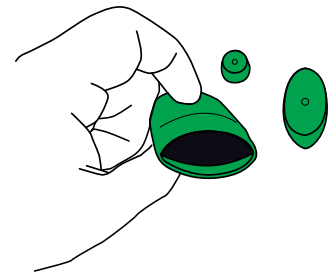


3

Take a deep breath before exhaling into the the sampling device, through the large opening. Repeat twelve times before putting the caps back on.

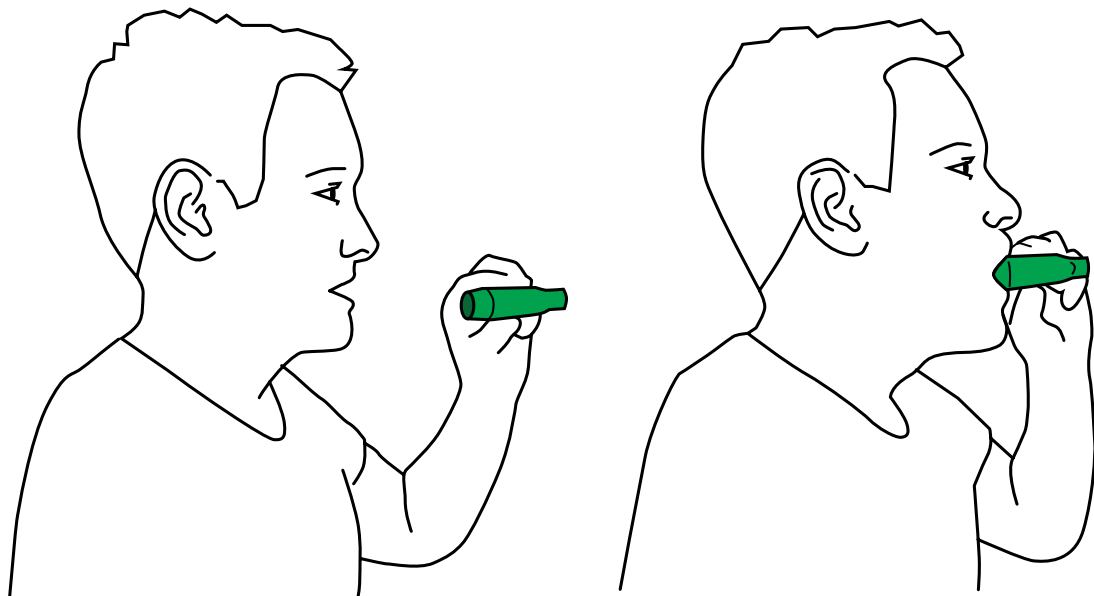



4

Take the barcode from the chain of custody form and stick it on to the sample device.

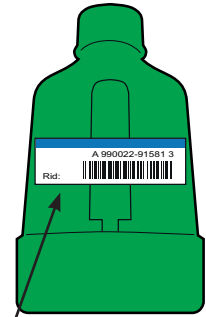


5

Put the sample device into the provided protective bag. Seal the bag with the small sticker that came with the bag, across the joint and over the perforation.

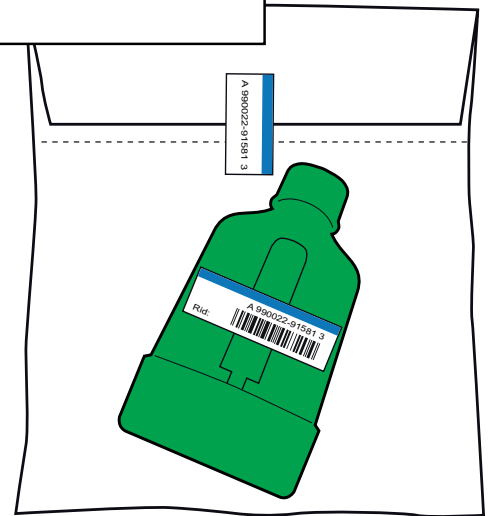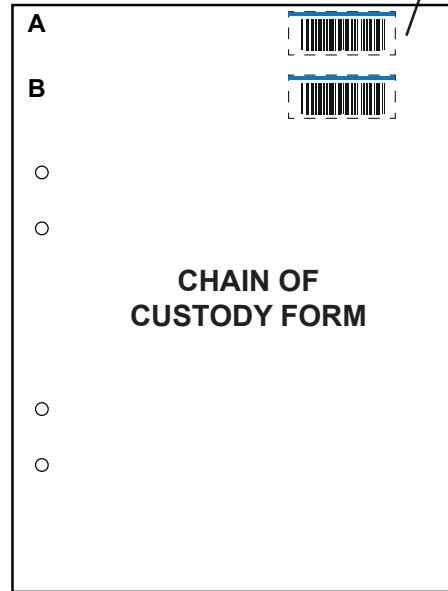

6

Fill out the chain of custody form and put the sealed-shut protective bag in an envelope and send it to the appointed laboratory.

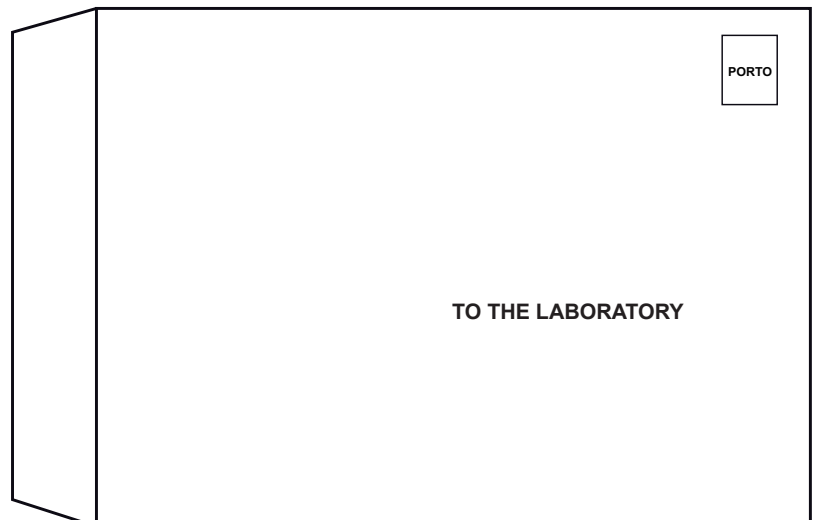

Munkplast AB  
Hållnäsgratan 6  
SE-752 28 Uppsala  
Sweden



Part S Modos AB  
Hållnäsgratan 6  
SE-752 28 Uppsala  
Sweden