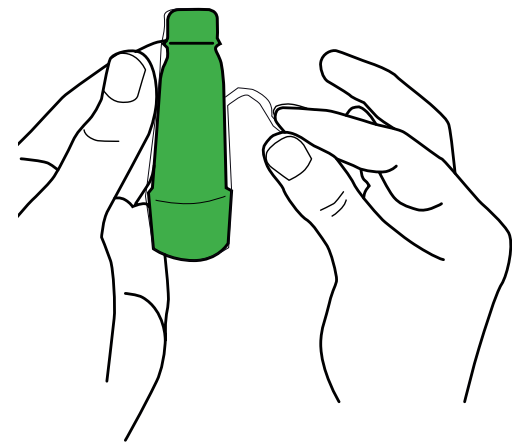
Instructions

It's simply breathtaking

Svenska

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

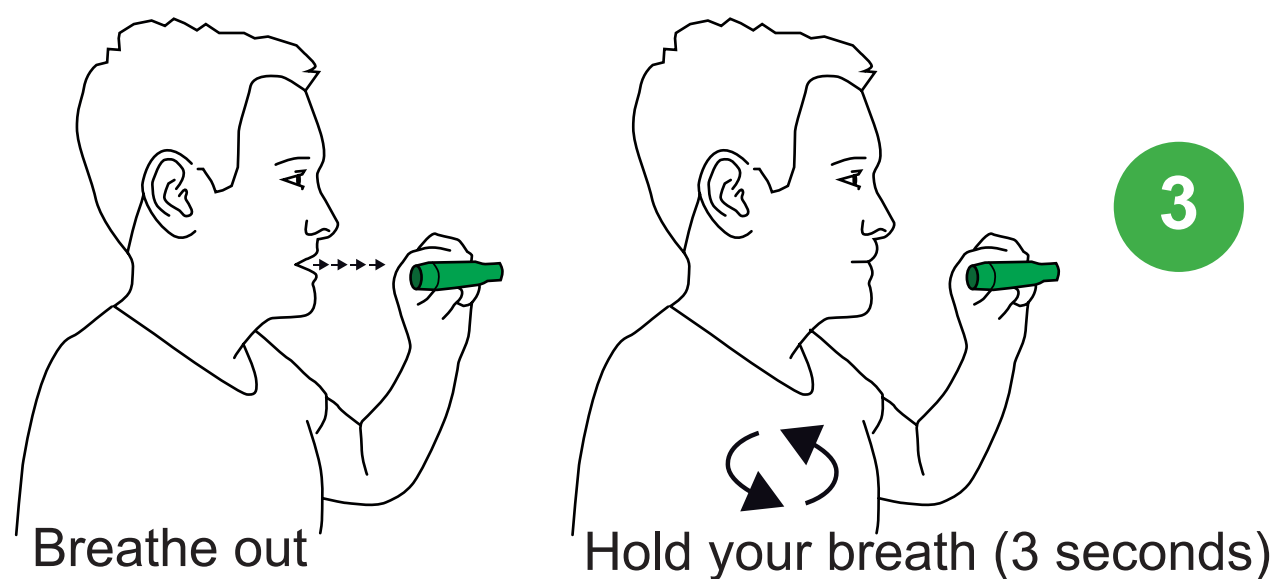
- 3 Prepare for a sample: Hold the Breath Explor device with the mouthpiece towards you, **but without breathing into it.** Breathe out to empty the lungs. Hold your breath for three seconds. Next, breathe in deeply.

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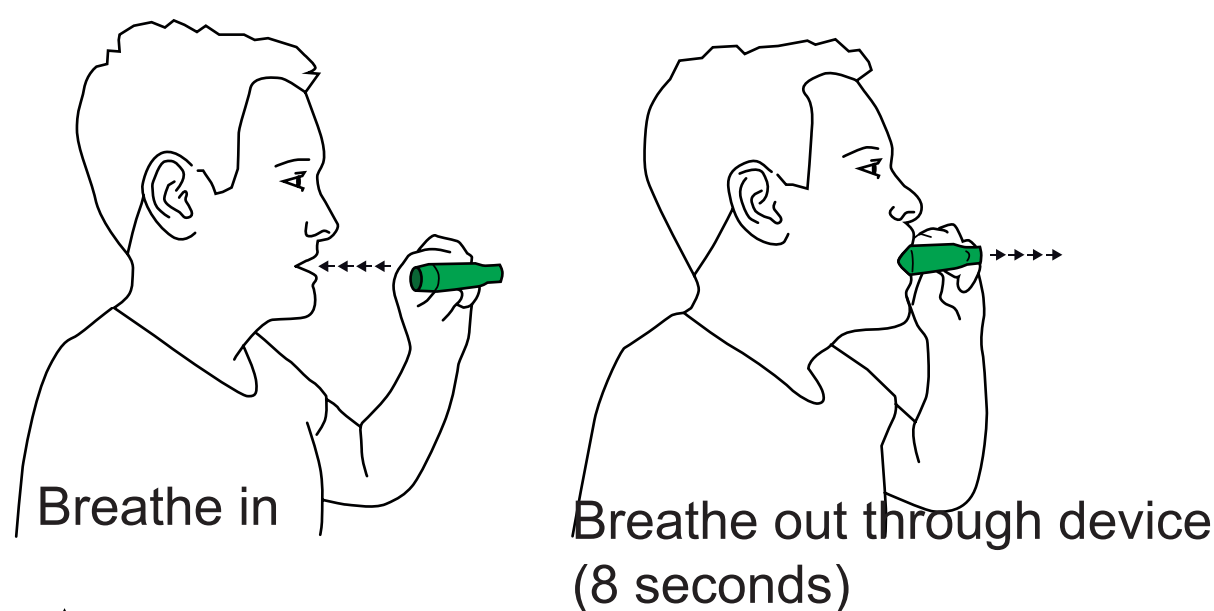
Leave the sample: Now, close your lips around the mouthpiece and breathe out through the device for eight seconds. **Remove the device from your mouth before inhaling as normal.** Repeat the procedure (exhale, pause 3 seconds, inhale, exhale through device) 10–12 times. If you feel dizzy, take a break and breathe normally for a few breaths. Once you are done, put the caps back on the Breath Explor device.

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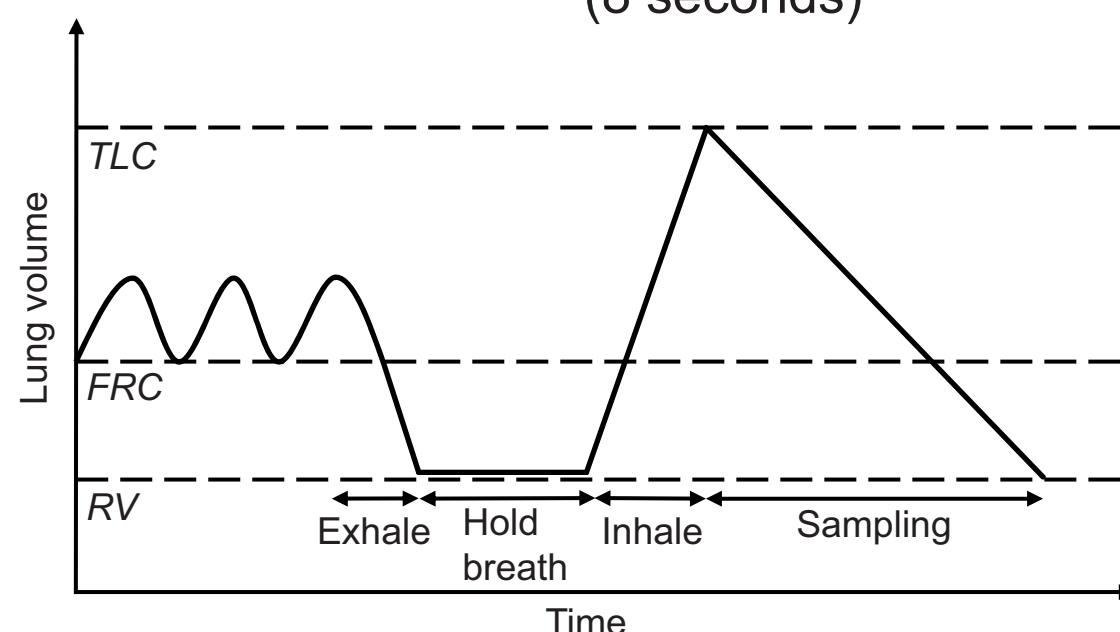
For guided assistance through the breath-sampling procedure, **please download our free app Breath Explor Operator Guide** from the App Store or Google Play. The app will help with timing and recording the breaths.

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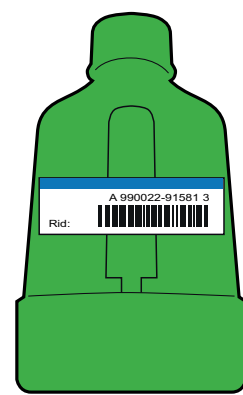- 4 Take one of the three barcode stickers from the Chain of Custody form and apply it to the sampling device.

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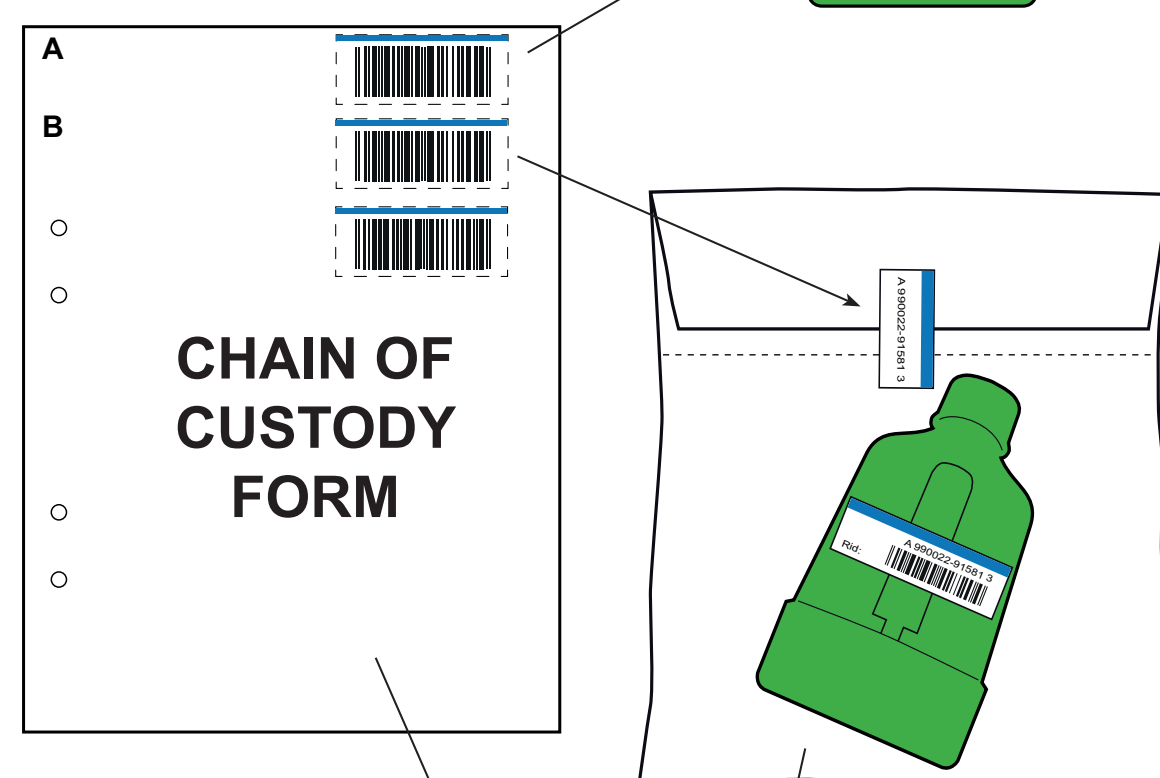
- 5 Put the sampling device into the provided protective bag. Seal the bag shut using another of barcode sticker from the Chain of Custody form. Make sure that the sticker both **closes the bag** and **covers the perforation.**

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- 6 Store the sample in a freezer until it is time to send the sample off.

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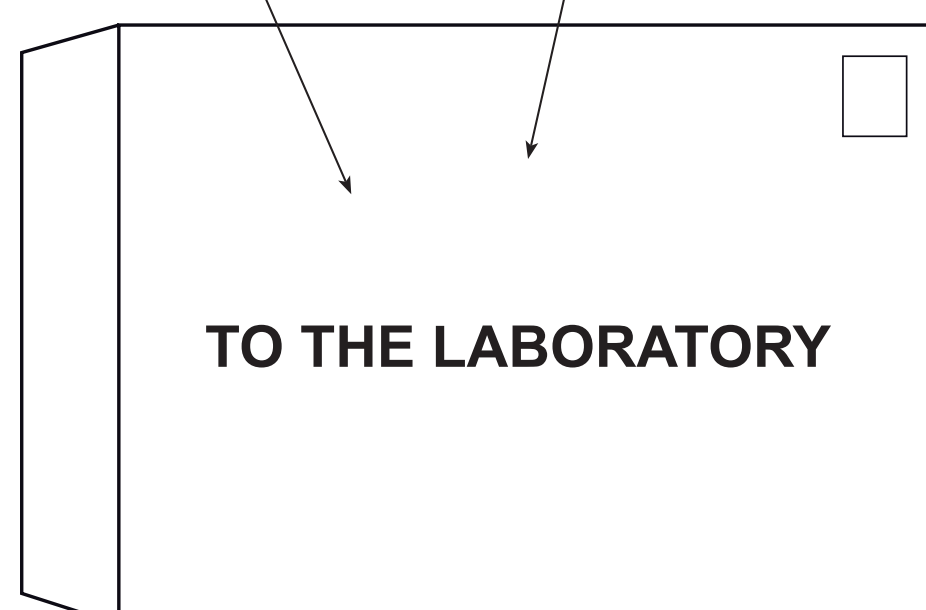- 7 Apply the third and last barcode sticker to the Chain of Custody form. Fill out the form. Put the Chain of Custody form and the sealed-shut protective bag with the Breath Explor in an envelope and send it to the appointed laboratory.

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For more information, please contact us at info@breathexplor.com.